

To Whom It May Concern:

Commission expires:

I hereby attest that Michael T. Maloy, Jr. is a Certified Intoxilyzer Quality Assurance Specialist for the Lake Havasu City Police Department.

I certify that the attached documentation of Standard Quality Assurance Procedures and/or Standard Calibration Check Procedure are full, true and correct copy of the original.

STATE OF ARIZONA)			
) ss			
COUNTY OF MOHAVE)			
Carlo and a state of the state	ore me this		A . 1 C	
Subscribed and sworn to bef	ore me this	_ day of	AUGUST.	
2020 by MI	CHAEL MALOY			
	ama pasay			
Chula 18	wite			
NOTARY PUBLIC	The state of the s			
0 10 5				
8-18-20				
DATE				

Area Code 928

CARLA DUARTE Notary Public - Arizona Mohave County My Comm. Expires Mar 29, 2021

Administration	855-4884	Fax #'s	Emergency 9-1-1
Business Office	855-1171	Administration 680-5430	Non-Emergency Dispatch/ 855-4111
Investigations	855-5775	Business Office 680-5431	
Patrol	855-0515	Investigations/Patrol 680-5432	

E-mail: police@lhcaz.gov

EXHIBIT G-2 THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. MALOY #181	AGENCY LUCAS
DATE 0818-1010	TIME 6409
INTOXILYZER SERIAL # 80-001873	LOCATION LILPO JAIL
(1. Ensure that gas tank is attached to instrument and	contains a standard alcohol concentration solution AC
Pour a standard alcohol concentration solution simulator. Ensure that a tight seal has been made.	AC, into a clean dry simulator and assemble the Turn on the simulator and allow temperature to reach 34°C ± FO START" dry control test or "W" for wet control test. After selection isAC.
DPS Form Exh G-2 (Rev 05-01)	

INTOXILYZER 8000 Location: LHCPD

Serial Number: 80-001873 Core Version: 8105.48

08/18/2.020

04:09:30

Standard Lot#: 35019100A1

Last Changed By: M. MALOY JR #281

QAS: M. MALOY JR #281 LAKE HAVASU CITY PD

Operator: M. MALOY JR #281

LAKE MAVASU CITY PD

Test	g/210L	Time ·
Air Blank	0.000	. 04:10:04
0.100 Cal Check	0.098	04:10:25
Air Blank	0.000	04:10:54

Preventative Maintenance Performed -- Timer Reset

EXHIBIT G-4 THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST M. MALOY #281	AGENCY L	MCPD	
DATE 08-18-2020	TIME	0411	
INTOXILYZER SERIAL # 80-001873	LOCATION	LHCPD	JAIL >
(1. Display Reads "PUSH BUTTON TO START"			
DIAGNOSTIC TESTS () 1. Clock time check. () 2. Date check.			
OPERATIONAL TESTS () 1. Alcohol-free subject test result 0. Do AC () 2. Error recognition logic system functioning. Not a Successfully Completed Test Sequence () 3. Proper sample recognition system. Not a Successfully Completed Test Sequence Deficient sample printed. () 4. Standard Calibration Check standard 0. Loo	ce printed	098	AC.
Instrument is operating properly and accurately. YES	<pre></pre>		
COMMENTS			
SIGNATURE Michael Maley S.	#181		
DPS Form Exh G-4 (Rev 05-01)			

INTOXILYZER 8000 Location: LHCPD

Serial Number: 80-001873 Core Version: 8105.48

08/18/2020

04:11:34

Standard Lot#: 35019100A1	
Last Changed By: M. MALOY JR	#281
QAS: M. MALOY JR LAKE HAVASU CITY PD	#281
Operator: M. MALOY JR LAKE HAVASU CITY PD	#281

Subject: SQAP, TEST DOB: 01/01/2000

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

<u>Test</u> ·	g/210L	Time
Air Blank	0.000	04:12:30
Diagnostic Test	Pass	04:13:05
Air Blank	0.000	04:13:33
0.100 Cal Check	0.098	04:13:54
Air Blank	0.000	04:14:23
Subject Test	0.000	04:14:42
Air Blank	ABT*	04:14:48
Air Blank	IPS**	04:14:57

^{*}Sequence Aborted

^{**}Improper Sample

INTOXILYZER 8000 Location: LHCPD

Serial Number: 80-001873 Core Version: 8105.48

08/18/2020 04:15:31

Standard Lot#: 35019100A1
Last Changed By: M. MALOY JR #281

QAS: M. MALOY JR #281
LAKE HAVASU CITY PD

Operator: M. MALOY JR #281
LAKE HAVASU CITY PD

Subject: SQAP, TEST DOB: 01/01/2000

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	04:16:20
Diagnostic Test	Pass	04:16:55
Air Blank	0.000	04:17:23
0.100 Cal Check	0.098	04:17:44
Air Blank	0.000	04:18:13
Subject Test	0.000*	04:21:20
Air Blank	0.000	04:21:48
Air Blank	0.000	04:22:15
Subject Test	XXX**	04:22:29
Air Blank .	RFI***	04:22:45

*Deficient Sample

**Invalid Sample

Begin new deprivation period

***RFI Detect

EXHIBIT G-2 THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. MALOY #281	_AGENCY	LHCAD	
DATE 08-18-2020	_ TIME	0423	
INTOXILYZER SERIAL # 80-001873			JAIL
($$			S
Pour a standard alcohol concentration solution simulator. Ensure that a tight seal has been made. 0.2°C 2. Intoxilyzer 8000 display reads "PUSH BUTTON". Solvent "D" for made press ENTER. 4. Air blank completed. 5. Calibration check completed. Test results 0.091 6. Air blank completed. 7. Remove printed record. Attach the record to the complete simulation.	AC, i Turn on the sir FO START" dry control test AC.	nulator and allo	w temperature to reach 34°C ±
SIGNATURE Mychael Maloy gr	#281		
DPS Form Exh G-2 (Rev 05-01)			

INTOXILYZER 8000 Location: LHCPD

Serial Number: 80-001873 Core Version: 8105.48

08/18/2020 04:23:44

Standard Lot#: 35019100A1

Last Changed By: M. MALOY JR #281

_____.

QAS: M. MALOY JR #281

LAKE HAVASU CITY PD

Operator: M. MALOY JR #281

LAKE HAVASU CITY PD

Test '	g/210L	Time
Air Blank	0.000	04:24:15
0.100 Cal Check	0.098	04:24:36
Air Blank	0.000	04:25:06

Preventative Maintenance Performed -- Timer Reset