



POLICE DEPARTMENT  
**LAKE HAVASU CITY**  
 2360 McCULLOCH BOULEVARD NORTH  
 LAKE HAVASU CITY, ARIZONA 86403-5947

To Whom It May Concern:

I hereby attest that Michael T. Maloy, Jr. is a Certified Intoxilyzer Quality Assurance Specialist for the Lake Havasu City Police Department.

I certify that the attached documentation of Standard Quality Assurance Procedures and/or Standard Calibration Check Procedure are full, true and correct copy of the original.

Michael Maloy Jr #281  
 Signature

STATE OF ARIZONA        )  
   ) ss  
 COUNTY OF MOHAVE     )


Subscribed and sworn to before me this 17<sup>th</sup> day of January.

2021 by Michael Maloy

Rebecca Duthie  
 NOTARY PUBLIC

1/17/2021  
 DATE

Commission expires: 12/7/2023

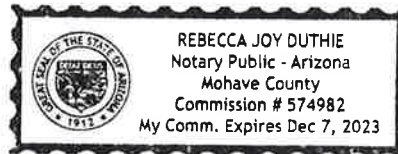

Area Code 928

Administration ..... 855-4884	Fax #'s	Emergency ..... 9-1-1
Business Office ..... 855-1171	Administration ..... 680-5430	Non-Emergency Dispatch/ ..... 855-4111
Investigations ..... 855-5775	Business Office ..... 680-5431	Havasu Silent Witness ..... 854-TIPS
Patrol ..... 855-0515	Investigations/Patrol ..... 680-5432	

E-mail: police@lhcaz.gov

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. MALOY #281 AGENCY LNCPD

DATE 01-17-2021 TIME 0911

INTOXILYZER SERIAL # 80-003356 LOCATION S16N8

- ( ✓ ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- ( ✓ ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- ( ✓ ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- ( ✓ ) 4. Air blank completed.
- ( ✓ ) 5. Calibration check completed. Test results 0. 099 AC.
- ( ✓ ) 6. Air blank completed.
- ( ✓ ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Michael Maloy #281

281

INTOXILYZER 8000  
Location: CHANNEL SIGN 8  
Serial Number: 80-003356  
Core Version: 8105.48  
01/17/2021 09:11:51

-----  
Standard Lot#: 35019100A1  
Last Changed By: M. MALOY JR #281  
-----

QAS: M. MALOY JR #281  
LAKE HAVASU CITY PD  
-----

Operator: M. MALOY JR #281  
LAKE HAVASU CITY PD  
-----

Test	g/210L	Time
Air Blank	0.000	09:12:25
0.100 Cal Check	0.099	09:12:46
Air Blank	0.000	09:13:16

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST M. MALOY #281 AGENCY LHCPD

DATE 01-17-2021 TIME 0915

INTOXILYZER SERIAL # 80-003356 LOCATION SIBN 8

(  ) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (  ) 1. Clock time check.
- (  ) 2. Date check.

OPERATIONAL TESTS

- (  ) 1. Alcohol-free subject test result 0. 000 AC.
- (  ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- (  ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- (  ) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

---



---



---



---

SIGNATURE Michael Maloy #281

INTOXILYZER 8000  
 Location: CHANNEL SIGN 8  
 Serial Number: 80-003356  
 Core Version: 8105.48  
 01/17/2021

09:15:29

---

Standard Lot#: 35019100A1  
 Last Changed By: M. MALOY JR #281

---

QAS: M. MALOY JR #281  
 LAKE HAVASU CITY PD

---

Operator: M. MALOY JR #281  
 LAKE HAVASU CITY PD

---

Subject: SQAP,TEST  
 DOB: 01/01/2000  
 Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	09:16:19
Diagnostic Test	Pass	09:16:54
Air Blank	0.000	09:17:21
0.100 Cal Check	0.099	09:17:42
Air Blank	0.000	09:18:11
Subject Test	0.000	09:18:36
Air Blank	ABT*	09:18:42
Air Blank	IPS**	09:18:50

\*Sequence Aborted

\*\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: CHANNEL SIGN 8  
Serial Number: 80-003356  
Core Version: 8105.48  
01/17/2021

09:19:36

---

Standard Lot#: 35019100A1  
Last Changed By: M. MALOY JR #281

---

QAS: M. MALOY JR #281  
LAKE HAVASU CITY PD

---

Operator: M. MALOY JR #281  
LAKE HAVASU CITY PD

---

Subject: SQAP,TEST  
DOB: 01/01/2000  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	09:20:23
Diagnostic Test	Pass	09:20:59
Air Blank	0.000	09:21:26
0.100 Cal Check	0.099	09:21:47
Air Blank	0.000	09:22:16
Subject Test	0.000*	09:25:24
Air Blank	0.000	09:25:51
Air Blank	0.000	09:26:19
Subject Test	XXX**	09:26:33
Air Blank	RFI***	09:26:43

\*Deficient Sample  
\*\*Invalid Sample  
    Begin new deprivation period  
\*\*\*RFI Detect

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. MALOY #281 AGENCY LHCPD

DATE 01-17-2021 TIME 0927

INTOXILYZER SERIAL # 80-003356 LOCATION S16N8

- (  ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (  ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (  ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (  ) 4. Air blank completed.
- (  ) 5. Calibration check completed. Test results 0. 097 AC.
- (  ) 6. Air blank completed.
- (  ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Michael Maloy Jr #281

281

INTOXILYZER 8000  
Location: CHANNEL SIGN 8  
Serial Number: 80-003356  
Core Version: 8105.48  
01/17/2021 09:27:53

-----  
Standard Lot#: 35019100A1  
Last Changed By: M. MALOY JR #281  
-----

QAS: M. MALOY JR #281  
LAKE HAVASU CITY PD  
-----

Operator: M. MALOY JR #281  
LAKE HAVASU CITY PD  
-----

Test	g/210L	Time
Air Blank	0.000	09:28:23
0.100 Cal Check	0.097	09:28:44
Air Blank	0.000	09:29:13

Preventative Maintenance  
Performed -- Timer Reset