



POLICE DEPARTMENT

LAKE HAVASU CITY

2360 McCULLOCH BOULEVARD NORTH
LAKE HAVASU CITY, ARIZONA 86403-5947

To Whom It May Concern:

I hereby attest that Josh Sondrol is a Certified Intoxilyzer Quality Assurance Specialist for the Lake Havasu City Police Department.

I certify that the attached documentation of Standard Quality Assurance Procedures and/or Standard Calibration Check Procedure are full, true and correct copy of the original.

[Handwritten Signature]

Signature

STATE OF ARIZONA)
) ss
COUNTY OF MOHAVE)


Subscribed and sworn to before me this 24 day of July.

2019 by Josh Sondrol

[Handwritten Signature]
NOTARY PUBLIC

July 24, 2019
DATE

Commission expires:

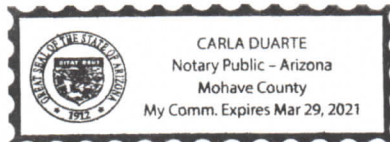

Area Code 928

Administration 855-4884
Business Office 855-1171
Investigations 855-5775
Patrol 855-0515

Fax #'s
Administration 680-5430
Business Office 680-5431
Investigations/Patrol 680-5432

Emergency 9-1-1
Non-Emergency Dispatch/ 855-4111
Havasu Silent Witness 854-TIPS

E-mail: police@lhcaz.gov

Exhibit G-2. Standard Calibration Check Procedure, Intoxilyzer Model 8000

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)
ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J Sondra 187 AGENCY LNC PD

DATE 7/20/19 TIME 1038

INTOXILYZER SERIAL # 80-003255 LOCATION JAIL

() 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .150 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34° C ± 0.2° C

() 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"

() 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.

() 4. Air blank completed.

() 5. Calibration check completed. Test results 0. 099 AC.

() 6. Air blank completed.

() 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J Sondra 187

DPS Form Exh G-2 (Rev 05-01)

Historical Note

New Exhibit G-2 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

INTOXILYZER 8000
Location: JAIL
Serial Number: 80-003355
Core Version: 8105.48
07/20/2019 10:38:04

Standard Lot#: 13518100A1
Last Changed By: J. SONDROL #187

QAS: J. SONDROL #187
LAKE HAVASU CITY PD

Operator: J. SONDROL #187
LAKE HAVASU CITY PD

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 10:38:34 |
| 0.100 Cal Check | 0.099 | 10:38:55 |
| Air Blank | 0.000 | 10:39:24 |

Preventative Maintenance
Performed -- Timer Reset