



POLICE DEPARTMENT  
**LAKE HAVASU CITY**

2360 McCULLOCH BOULEVARD NORTH  
LAKE HAVASU CITY, ARIZONA 86403-5947

To Whom It May Concern:

I hereby attest that Michael T. Maloy, Jr. is a Certified Intoxilyzer Quality Assurance Specialist for the Lake Havasu City Police Department.

I certify that the attached documentation of Standard Quality Assurance Procedures and/or Standard Calibration Check Procedure are full, true and correct copy of the original.

Michael Maloy Jr #281  
Signature

STATE OF ARIZONA            )  
  ) ss  
COUNTY OF MOHAVE        )


Subscribed and sworn to before me this 31 day of July

2021 by Ashley Rae Bonds

Ashley Rae Bonds  
NOTARY PUBLIC

7/31/21  
DATE

Commission expires: May 29, 2025

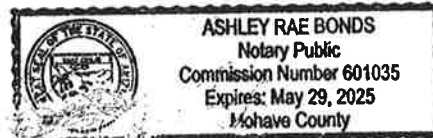

Area Code 928

Administration ..... 855-4884	Fax #'s	Emergency ..... 9-1-1
Business Office ..... 855-1171	Administration ..... 680-5430	Non-Emergency Dispatch/ ..... 855-4111
Investigations ..... 855-5775	Business Office ..... 680-5431	Havasu Silent Witness ..... 854-TIPS
Patrol ..... 855-0515	Investigations/Patrol ..... 680-5432	

E-mail: police@lhcaz.gov

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. MALDY #281 AGENCY LHCPD  
DATE 07-31-2021 TIME 0033  
INTOXILYZER SERIAL # 80-003356 LOCATION S16N B

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 098 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Michael Maldy #281

281

INTOXILYZER 8000  
Location: CHANNEL SIGN 8  
Serial Number: 80-003356  
Core Version: 8105.48  
07/31/2021 00:33:55

-----  
Standard Lot#: 35019100A1  
Last Changed By: M. MALOY JR #281  
-----

QAS: M. MALOY JR #281  
LAKE HAVASU CITY PD  
-----

Operator: M. MALOY JR #281  
LAKE HAVASU CITY PD  
-----

Test	g/210L	Time
Air Blank	0.000	00:34:27
0.100 Cal Check	0.098	00:34:48
Air Blank	0.000	00:35:18

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST M. MALOY #281 AGENCY LHCPS

DATE 07-31-2021 TIME 0036

INTOXILYZER SERIAL # 80-003356 LOCATION S16N8

(  ) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

(  ) 1. Clock time check.

(  ) 2. Date check.

OPERATIONAL TESTS

(  ) 1. Alcohol-free subject test result 0. 000 AC.

(  ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed

(  ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.

(  ) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE Michael Maloy Jr #281

281

INTOXILYZER 8000  
Location: CHANNEL SIGN 8  
Serial Number: 80-003356  
Core Version: 8105.48  
07/31/2021

00:36:31

---

Standard Lot#: 35019100A1  
Last Changed By: M. MALOY JR #281

---

QAS: M. MALOY JR #281  
LAKE HAVASU CITY PD

---

Operator: M. MALOY JR #281  
LAKE HAVASU CITY PD

---

Subject: SQAP,TEST  
DOB: 01/01/2000  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	00:37:18
Diagnostic Test	Pass	00:37:53
Air Blank	0.000	00:38:21
0.100 Cal Check	0.098	00:38:42
Air Blank	0.000	00:39:11
Subject Test	0.000	00:39:38
Air Blank	ABT*	00:39:45
Air Blank	IPS**	00:39:53

\*Sequence Aborted  
\*\*Improper Sample

**Not a Successfully Completed Test Sequence**

281

INTOXILYZER 8000  
Location: CHANNEL SIGN 8  
Serial Number: 80-003356  
Core Version: 8105.48  
07/31/2021 00:40:17

---

Standard Lot#: 35019100A1  
Last Changed By: M. MALOY JR #281

---

QAS: M. MALOY JR #281  
LAKE HAVASU CITY PD

---

Operator: M. MALOY JR #281  
LAKE HAVASU CITY PD

---

Subject: SQAP,TEST  
DOB: 01/01/2000  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	00:41:04
Diagnostic Test	Pass	00:41:39
Air Blank	0.000	00:42:07
0.100 Cal Check	0.098	00:42:28
Air Blank	0.000	00:42:58
Subject Test	0.000*	00:46:05
Air Blank	0.000	00:46:32
Air Blank	0.000	00:47:00
Subject Test	XXX**	00:47:15
Air Blank	RFI***	00:47:28

\*Deficient Sample  
\*\*Invalid Sample  
Begin new deprivation period  
\*\*\*RFI Detect

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. MALOY #281 AGENCY LHCPO

DATE 07-31-2021 TIME 0648

INTOXILYZER SERIAL # 80-003356 LOCATION SIGN 8

- (  ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (  ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (  ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (  ) 4. Air blank completed.
- (  ) 5. Calibration check completed. Test results 0. 097 AC.
- (  ) 6. Air blank completed.
- (  ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Michael Maloy #281

281

INTOXILYZER 8000  
Location: CHANNEL SIGN 8  
Serial Number: 80-003356  
Core Version: 8105.48  
07/31/2021 00:48:05

-----  
Standard Lot#: 35019100A1  
Last Changed By: M. MALOY JR #281  
-----

QAS: M. MALOY JR #281  
LAKE HAVASU CITY PD  
-----

Operator: M. MALOY JR #281  
LAKE HAVASU CITY PD  
-----

Test	g/210L	Time
Air Blank	0.000	00:48:36
0.100 Cal Check	0.097	00:48:56
Air Blank	0.000	00:49:26

Preventative Maintenance  
Performed -- Timer Reset