



POLICE DEPARTMENT  
**LAKE HAVASU CITY**

2360 McCULLOCH BOULEVARD NORTH  
LAKE HAVASU CITY, ARIZONA 86403-5947

To Whom It May Concern:

I hereby attest that Michael T. Maloy, Jr. is a Certified Intoxilyzer Quality Assurance Specialist for the Lake Havasu City Police Department.

I certify that the attached documentation of Standard Quality Assurance Procedures and/or Standard Calibration Check Procedure are full, true and correct copy of the original.

Michael Maloy Jr #281  
Signature

STATE OF ARIZONA        )  
  ) ss  
COUNTY OF MOHAVE    )


Subscribed and sworn to before me this 17<sup>th</sup> day of June.

2021 by Jennifer Lentz

Jennifer Lentz  
NOTARY PUBLIC

6/17/2021  
DATE

Commission expires:

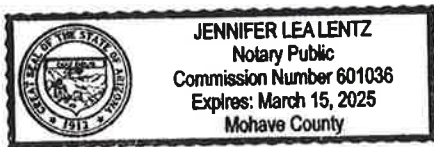

Area Code 928

Administration ..... 855-4884  
Business Office ..... 855-1171  
Investigations ..... 855-5775  
Patrol ..... 855-0515

Fax #'s  
Administration ..... 680-5430  
Business Office ..... 680-5431  
Investigations/Patrol ..... 680-5432

Emergency ..... 9-1-1  
Non-Emergency Dispatch/ ..... 855-4111  
Havasu Silent Witness ..... 854-TIPS

E-mail: police@lhcaz.gov

281

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. MALDY #281 AGENCY LHCPO

DATE 06-17-2021 TIME 0012

INTOXILYZER SERIAL # 80-003355 LOCATION LHCPO JAIL

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 099 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Michael Maloy #281

281

INTOXILYZER 8000  
Location: JAIL  
Serial Number: 80-003355  
Core Version: 8105.48  
06/17/2021 00:12:27

-----  
Standard Lot#: 35019100A1  
Last Changed By: M. MALOY JR #281  
-----

QAS: M. MALOY JR #281  
LAKE HAVASU CITY PD  
-----

Operator: M. MALOY JR #281  
LAKE HAVASU CITY PD  
-----

Test	g/210L	Time
Air Blank	0.000	00:13:00
0.100 Cal Check	0.099	00:13:21
Air Blank	0.000	00:13:51

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST M. MALOY #281 AGENCY LHCPS

DATE 06-17-2021 TIME 0015

INTOXILYZER SERIAL # 80-003355 LOCATION LHCPS JAIL

(  ) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

(  ) 1. Clock time check.

(  ) 2. Date check.

OPERATIONAL TESTS

(  ) 1. Alcohol-free subject test result 0. 000 AC.

(  ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed

(  ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.

(  ) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 100 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE Michael Maloy Jr #281

281

INTOXILYZER 8000  
Location: JAIL  
Serial Number: 80-003355  
Core Version: 8105.48  
06/17/2021 00:15:37

---

Standard Lot#: 35019100A1  
Last Changed By: M. MALOY JR #281

---

QAS: M. MALOY JR #281  
LAKE HAVASU CITY PD

---

Operator: M. MALOY JR #281  
LAKE HAVASU CITY PD

---

Subject: SQAP,TEST  
DOB: 01/01/2000  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	00:16:28
Diagnostic Test	Pass	00:17:03
Air Blank	0.000	00:17:30
0.100 Cal Check	0.100	00:17:51
Air Blank	0.000	00:18:21
Subject Test	0.000	00:18:52
Air Blank	ABT*	00:18:58
Air Blank	IPS**	00:19:07

\*Sequence Aborted  
\*\*Improper Sample

**Not a Successfully Completed Test Sequence**

281

INTOXILYZER 8000  
Location: JAIL  
Serial Number: 80-003355  
Core Version: 8105.48  
06/17/2021

00:19:48

---

Standard Lot#: 35019100A1  
Last Changed By: M. MALOY JR #281

---

QAS: M. MALOY JR #281  
LAKE HAVASU CITY PD

---

Operator: M. MALOY JR #281  
LAKE HAVASU CITY PD

---

Subject: SQAP,TEST  
DOB: 01/01/2000  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	00:20:36
Diagnostic Test	Pass	00:21:11
Air Blank	0.000	00:21:38
0.100 Cal Check	0.099	00:21:59
Air Blank	0.000	00:22:29
Subject Test	0.000*	00:25:36
Air Blank	0.000	00:26:03
Air Blank	0.000	00:26:31
Subject Test	XXX**	00:26:46
Air Blank	RFI***	00:26:59

\*Deficient Sample  
\*\*Invalid Sample  
    Begin new deprivation period  
\*\*\*RFI Detect

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. MALOY #281 AGENCY LHCPS

DATE 06-17-2021 TIME 0027

INTOXILYZER SERIAL # 80-003355 LOCATION LHCPS JAIL

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 100 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Michael Maloy #281

281

INTOXILYZER 8000

Location: JAIL

Serial Number: 80-003355

Core Version: 8105.48

06/17/2021

00:27:56

-----  
Standard Lot#: 35019100A1

Last Changed By: M. MALOY JR #281  
-----

QAS: M. MALOY JR #281

LAKE HAVASU CITY PD  
-----

Operator: M. MALOY JR #281

LAKE HAVASU CITY PD  
-----

Test	g/210L	Time
Air Blank	0.000	00:28:27
0.100 Cal Check	0.100	00:28:48
Air Blank	0.000	00:29:18

Preventative Maintenance  
Performed -- Timer Reset