



POLICE DEPARTMENT  
**LAKE HAVASU CITY**  
 2360 McCULLOCH BOULEVARD NORTH  
 LAKE HAVASU CITY, ARIZONA 86403-5947

To Whom It May Concern:

I hereby attest that Michael T. Maloy, Jr. is a Certified Intoxilyzer Quality Assurance Specialist for the Lake Havasu City Police Department.

I certify that the attached documentation of Standard Quality Assurance Procedures and/or Standard Calibration Check Procedure are full, true and correct copy of the original.

Michael Maloy Jr  
 Signature

STATE OF ARIZONA        )  
   ) ss  
 COUNTY OF MOHAVE     )


Subscribed and sworn to before me this 18 day of JUNE.

2020 by MICHAEL MALOY

Carla Duarte  
 NOTARY PUBLIC

6-18-20  
 DATE

Commission expires:

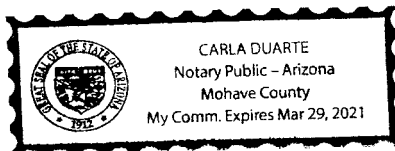

Area Code 928

Administration ..... 855-4884  
 Business Office ..... 855-1171  
 Investigations ..... 855-5775  
 Patrol ..... 855-0515

Fax #'s  
 Administration ..... 680-5430  
 Business Office ..... 680-5431  
 Investigations/Patrol ..... 680-5432

Emergency ..... 9-1-1  
 Non-Emergency Dispatch/ ..... 855-4111  
 Havasu Silent Witness ..... 854-TIPS

E-mail: [police@lhcaz.gov](mailto:police@lhcaz.gov)

281

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. MALOY AGENCY LHCPD  
DATE 06-18-2020 TIME 0429  
INTOXILYZER SERIAL # 80-003356 LOCATION SIGN 8

- (  ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (  ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (  ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (  ) 4. Air blank completed.
- (  ) 5. Calibration check completed. Test results C 0097 AC.
- (  ) 6. Air blank completed.
- (  ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Michael Maloy Jr

281

INTOXILYZER 8000

Location: CHANNEL SIGN 8

Serial Number: 80-003356

Core Version: 8105.48

06/18/2020

04:29:14

-----  
Standard Lot#: 35019100A1

Last Changed By: M. MALOY JR #281  
-----

QAS: M. MALOY JR #281

LAKE HAVASU CITY PD  
-----

Operator: M. MALOY JR #281

LAKE HAVASU CITY PD  
-----

Test	g/210L	Time
Air Blank	0.000	04:29:46
0.100 Cal Check	0.097	04:30:07
Air Blank	0.000	04:30:37

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST M. MALDY AGENCY LULPD

DATE 06-18-2020 TIME 0433

INTOXILYZER SERIAL # 80-003356 LOCATION SIGN B

(  ) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (  ) 1. Clock time check.
- (  ) 2. Date check.

OPERATIONAL TESTS

- (  ) 1. Alcohol-free subject test result 0. 000 AC.
- (  ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- (  ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- (  ) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 090 AC.

Instrument is operating properly and accurately. YES  NO \_\_\_\_\_

COMMENTS

---



---



---



---

SIGNATURE Michael Maldy Jr

281

INTOXILYZER 8000  
Location: CHANNEL SIGN 8  
Serial Number: 80-003356  
Core Version: 8105.48  
06/18/2020 04:33:10

---

Standard Lot#: 35019100A1  
Last Changed By: M. MALOY JR #281

---

QAS: M. MALOY JR #281  
LAKE HAVASU CITY PD

---

Operator: M. MALOY JR #281  
LAKE HAVASU CITY PD

---

Subject: TEST,SQAP  
DOB: 01/01/2000  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	04:34:20
Diagnostic Test	Pass	04:34:55
Air Blank	0.000	04:35:23
0.100 Cal Check	0.098	04:35:44
Air Blank	0.000	04:36:13
Subject Test	0.000	04:36:28
Air Blank	ABT*	04:36:35
Air Blank	IPS**	04:36:43

\*Sequence Aborted  
\*\*Improper Sample

**Not a Successfully Completed Test Sequence**

281

INTOXILYZER 8000  
Location: CHANNEL SIGN 8  
Serial Number: 80-003356  
Core Version: 8105.48  
06/18/2020

04:37:04

---

Standard Lot#: 35019100A1  
Last Changed By: M. MALOY JR #281

---

QAS: M. MALOY JR #281  
LAKE HAVASU CITY PD

---

Operator: M. MALOY JR #281  
LAKE HAVASU CITY PD

---

Subject: SQAP,TEST  
DOB: 01/01/2000  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	04:37:56
Diagnostic Test	Pass	04:38:31
Air Blank	0.000	04:38:59
0.100 Cal Check	0.098	04:39:19
Air Blank	0.000	04:39:49
Subject Test	0.000*	04:42:56
Air Blank	0.000	04:43:24
Air Blank	0.000	04:43:51
Subject Test	XXX**	04:44:16
Air Blank	RFI***	04:44:23

\*Deficient Sample  
\*\*Invalid Sample  
Begin new deprivation period  
\*\*\*RFI Detect

**Not a Successfully Completed Test Sequence**

281

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. MALDY AGENCY LHCPO  
DATE 06-18-2020 TIME 0446  
INTOXILYZER SERIAL # 80-603356 LOCATION SIGN 8

- (  ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (  ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (  ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (  ) 4. Air blank completed.
- (  ) 5. Calibration check completed. Test results 0. 098 AC.
- (  ) 6. Air blank completed.
- (  ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Michael Maldy  
DPS Form Exh G-2 (Rev 05-01)

281

INTOXILYZER 8000  
Location: CHANNEL SIGN 8  
Serial Number: 80-003356  
Core Version: 8105.48  
06/18/2020 04:46:17

-----  
Standard Lot#: 35019100A1  
Last Changed By: M. MALOY JR #281  
-----

QAS: M. MALOY JR #281  
LAKE HAVASU CITY PD  
-----

Operator: M. MALOY JR #281  
LAKE HAVASU CITY PD  
-----

Test	g/210L	Time
Air Blank	0.000	04:46:47
0.100 Cal Check	0.098	04:47:08
Air Blank	0.000	04:47:38

Preventative Maintenance  
Performed -- Timer Reset