



POLICE DEPARTMENT
LAKE HAVASU CITY
2360 McCULLOCH BOULEVARD NORTH
LAKE HAVASU CITY, ARIZONA 86403-5947

To Whom It May Concern:

I hereby attest that Josh Sondrol is a Certified Intoxilyzer Quality Assurance Specialist for the Lake Havasu City Police Department.

I certify that the attached documentation of Standard Quality Assurance Procedures and/or Standard Calibration Check Procedure are full, true and correct copy of the original.

Josh Sondrol 187
Signature

STATE OF ARIZONA)
) ss
COUNTY OF MOHAVE)

Subscribed and sworn to before me this 20 day of MARCH.


2019 by JOSH SONDROL

Carla Duarte

NOTARY PUBLIC

3-20-19
DATE

Commission expires:

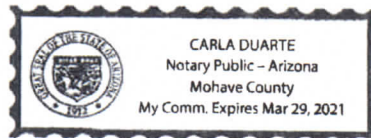

Area Code 928

Administration 855-4884
Business Office 855-1171
Investigations 855-5775
Patrol 855-0515

Fax #'s
Administration 680-5430
Business Office 680-5431
Investigations/Patrol 680-5432

Emergency 9-1-1
Non-Emergency Dispatch/ 855-4111
Havasu Silent Witness 854-TIPS

E-mail: police@lhcaz.gov

Exhibit G-2. Standard Calibration Check Procedure, Intoxilyzer Model 8000

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)
ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J SANDRAC 187 AGENCY LNC PD
DATE 3/20/19 TIME 0633
INTOXILYZER SERIAL # 80-003355 LOCATION JAIL

- 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC. into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34° C ± 0.2° C
- 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- 4. Air blank completed.
- 5. Calibration check completed. Test results 0. 101 AC.
- 6. Air blank completed.
- 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J Sandrac 187

DPS Form Exh G-2 (Rev 05-01)

Historical Note
New Exhibit G-2 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

Q187

INTOXILYZER 8000
Location: JAIL
Serial Number: 80-003355
Core Version: 8105.48
03/20/2019 06:33:28

Standard Lot#: 13917100A1
Last Changed By: J. SONDRÖL #187

QAS: J. SONDRÖL #187
LAKE HAVASU CITY PD

Operator: J. SONDRÖL #187
LAKE HAVASU CITY PD

Test	g/210L	Time
Air Blank	0.000	06:33:59
0.100 Cal Check	0.101	06:34:19
Air Blank	0.000	06:34:49

Preventative Maintenance
Performed -- Timer Reset

Exhibit G-4. Standard Quality Assurance Procedure Intoxilyzer, Model 8000

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST J SANDERSON 187 AGENCY LNC PD
DATE 3/20/19 TIME 0635
INTOXILYZER SERIAL # 80-003555 LOCATION JAIL

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
- () 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
- () 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- () 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- () 4. Standard Calibration Check standard 0. 100 AC. Result 0. 100 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS _____

SIGNATURE J Sanderson 187

DPS Form Exh G-4 (Rev 05-01)

Historical Note

New Exhibit G-4 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

0187

INTOXILYZER 8000
Location: JAIL
Serial Number: 80-003355
Core Version: 8105.48
03/20/2019

06:35:04

Standard Lot#: 13917100A1
Last Changed By: J. SONDROL #187

QAS: J. SONDROL #187
LAKE HAVASU CITY PD

Operator: J. SONDROL #187
LAKE HAVASU CITY PD

Subject: SQAP, TEST
DOB: 01/01/1959
Sex: M Weight: 123

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	06:35:53
Diagnostic Test	Pass	06:36:28
Air Blank	0.000	06:36:56
0.100 Cal Check	0.100	06:37:16
Air Blank	0.000	06:37:46
Subject Test	0.164*	06:40:53
Air Blank	0.000	06:41:26
Air Blank	0.000	06:41:53
Subject Test	XXX**	06:42:23
Air Blank	RFI***	06:42:32

*Deficient Sample
**Invalid Sample
 Begin new deprivation period
***RFI Detect

Not a Successfully Completed Test Sequence

9/18/17

INTOXILYZER 8000
Location: JAIL
Serial Number: 80-003355
Core Version: 8105.48
03/20/2019

06:43:05

Standard Lot#: 13917100A1
Last Changed By: J. SONDROL #187

QAS: J. SONDROL #187
LAKE HAVASU CITY PD

Operator: J. SONDROL #187
LAKE HAVASU CITY PD

Subject: SQAP,TEST
DOB: 01/01/1950
Sex: M Weight: 123

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	06:43:59
Diagnostic Test	Pass	06:44:34
Air Blank	0.000	06:45:02
0.100 Cal Check	0.099	06:45:22
Air Blank	0.000	06:45:52
Subject Test	0.000	06:46:14
Air Blank	ABT*	06:46:20
Air Blank	IPS**	06:46:28

*Sequence Aborted
**Improper Sample

Not a Successfully Completed Test Sequence

Exhibit G-2. Standard Calibration Check Procedure, Intoxilyzer Model 8000

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)
ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. SANDERSON 187 AGENCY LNC PD
DATE 3/20/19 TIME 0647
INTOXILYZER SERIAL # 80-003355 LOCATION JAIL

- (4) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34° C ± 0.2° C
- (4) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (4) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (4) 4. Air blank completed.
- (4) 5. Calibration check completed. Test results 0. 101 AC.
- (4) 6. Air blank completed.
- (4) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] 187

DPS Form Exh G-2 (Rev 05-01)

Historical Note
New Exhibit G-2 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

187

INTOXILYZER 8000
Location: JAIL
Serial Number: 80-003355
Core Version: 8105.48
03/20/2019 06:47:12

Standard Lot#: 13917100A1
Last Changed By: J. SONDROL #187

QAS: J. SONDROL #187
LAKE HAVASU CITY PD

Operator: J. SONDROL #187
LAKE HAVASU CITY PD

Test	g/210L	Time
Air Blank	0.000	06:47:41
0.100 Cal Check	0.101	06:48:01
Air Blank	0.000	06:48:31

Preventative Maintenance
Performed -- Timer Reset