



POLICE DEPARTMENT
LAKE HAVASU CITY
 2360 McCULLOCH BOULEVARD NORTH
 LAKE HAVASU CITY, ARIZONA 86403-5947

To Whom It May Concern:

I hereby attest that Josh Sondrol is a Certified Intoxilyzer Quality Assurance Specialist for the Lake Havasu City Police Department.

I certify that the attached documentation of Standard Quality Assurance Procedures and/or Standard Calibration Check Procedure are full, true and correct copy of the original.

Glenn
 Signature

STATE OF ARIZONA)
) ss
 COUNTY OF MOHAVE)


Subscribed and sworn to before me this 14 day of November.

2019 by Josh Sondrol
Carla Duarte

NOTARY PUBLIC

11-14-19
 DATE

Commission expires:

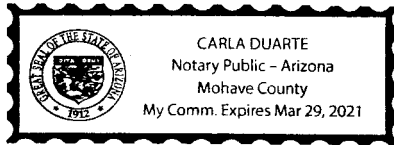

Area Code 928

Administration	855-4884	Fax #'s		Emergency	9-1-1
Business Office	855-1171	Administration	680-5430	Non-Emergency Dispatch/	855-4111
Investigations	855-5775	Business Office	680-5431	Havasu Silent Witness	854-TIPS
Patrol	855-0515	Investigations/Patrol	680-5432		

E-mail: police@lhcaz.gov

Exhibit G-2. Standard Calibration Check Procedure, Intoxilyzer Model 8000

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)
ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. SONDRA 187 AGENCY LINC PD
DATE 11/14/19 TIME 0859
INTOXILYZER SERIAL # 80-003355 LOCATION JAIL

(X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR

Pour a standard alcohol concentration solution _____ AC. into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34° C ± 0.2° C

- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0.100 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. SONDRA 187

DPS Form Exh G-2 (Rev 05-01)

Historical Note
New Exhibit G-2 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

INTOXILYZER 8000

Location: JAIL

Serial Number: 80-003355

Core Version: 8105.48

11/14/2019

08:59:55

Standard Lot#: 13518100A1

Last Changed By: J. SONDROL #187

QAS: J. SONDROL #187

LAKE HAVASU CITY PD

Operator: J. SONDROL #187

LAKE HAVASU CITY PD

Test	g/210L	Time
Air Blank	0.000	09:00:27
0.100 Cal Check	0.100	09:00:48
Air Blank	0.000	09:01:18

Preventative Maintenance
Performed -- Timer Reset

Exhibit G-4. Standard Quality Assurance Procedure Intoxilyzer, Model 8000

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST J SONDRA 187 AGENCY LNC PD
DATE 11/14/19 TIME 0905
INTOXILYZER SERIAL # 80-00355 LOCATION JAIL

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- 1. Clock time check.
- 2. Date check.

OPERATIONAL TESTS

- 1. Alcohol-free subject test result 0. 000 AC.
- 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- 4. Standard Calibration Check standard 0. 100 AC. Result 0. 100 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS _____

SIGNATURE J SONDRA 187

DPS Form Exh G-4 (Rev 05-01)

Historical Note

New Exhibit G-4 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

INTOXILYZER 8000
Location: JAIL
Serial Number: 80-003355
Core Version: 8105.48
11/14/2019

09:02:11

Standard Lot#: 13518100A1
Last Changed By: J. SONDROL #187

QAS: J. SONDROL #187
LAKE HAVASU CITY PD

Operator: J. SONDROL #187
LAKE HAVASU CITY PD

Subject: SQAP,TEST
DOB: 01/01/1950
Sex: M Weight: 123

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:02:57
Diagnostic Test	Pass	09:03:32
Air Blank	0.000	09:03:59
0.100 Cal Check	0.100	09:04:20
Air Blank	0.000	09:04:50
Subject Test	0.000	09:05:21
Air Blank	ABT*	09:05:27
Air Blank	IPS**	09:05:35

*Sequence Aborted

**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: JAIL
Serial Number: 80-003355
Core Version: 8105.48
11/14/2019

09:06:07

Standard Lot#: 13518100A1
Last Changed By: J. SONDROL #187

QAS: J. SONDROL #187
LAKE HAVASU CITY PD

Operator: J. SONDROL #187
LAKE HAVASU CITY PD

Subject: SQAP,TEST
DOB: 01/01/1950
Sex: M Weight: 123

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:06:52
Diagnostic Test	Pass	09:07:27
Air Blank	0.000	09:07:55
0.100 Cal Check	0.100	09:08:16
Air Blank	0.000	09:08:45
Subject Test	0.125*	09:11:53
Air Blank	0.000	09:12:26
Air Blank	0.000	09:12:53
Subject Test	XXX**	09:13:15
Air Blank	RFI***	09:13:22

*Deficient Sample

**Invalid Sample

Begin new deprivation period

***RFI Detect

Not a Successfully Completed Test Sequence

Exhibit G-2. Standard Calibration Check Procedure, Intoxilyzer Model 8000

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)
ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J SONDRA 187 AGENCY LNC PD
DATE 11/14/19 TIME _____

INTOXILYZER SERIAL # 80-003355 LOCATION JAIL

(1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34° C ± 0.2° C

(2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.

(4. Air blank completed.
(5. Calibration check completed. Test results 0. _____ AC.

(6. Air blank completed.
(7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Sondra 187

DPS Form Exh G-2 (Rev 05-01)

Historical Note
New Exhibit G-2 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

INTOXILYZER 8000

Location: JAIL

Serial Number: 80-003355

Core Version: 8105.48

11/14/2019

09:14:10

Standard Lot#: 13518100A1

Last Changed By: J. SONDROL #187

QAS: J. SONDROL #187

LAKE HAVASU CITY PD

Operator: J. SONDROL #187

LAKE HAVASU CITY PD

Test g/210L Time

Air Blank AMB* 09:14:40
Air Blank 0.000 09:15:08

*Ambient Fail

INTOXILYZER 8000

Location: JAIL

Serial Number: 80-003355

Core Version: 8105.48

11/14/2019

09:16:12

Standard Lot#: 13518100A1

Last Changed By: J. SONDRÖL #187

QAS: J. SONDRÖL #187

LAKE HAVASU CITY PD

Operator: J. SONDRÖL #187

LAKE HAVASU CITY PD

Test	g/210L	Time
Air Blank	0.000	09:16:41
0.100 Cal Check	0.102	09:17:02
Air Blank	0.000	09:17:32

Preventative Maintenance
Performed -- Timer Reset