



POLICE DEPARTMENT  
**LAKE HAVASU CITY**  
2360 McCULLOCH BOULEVARD NORTH  
LAKE HAVASU CITY, ARIZONA 86403-5947

To Whom It May Concern:

I hereby attest that Josh Sondrol is a Certified Intoxilyzer Quality Assurance Specialist for the Lake Havasu City Police Department.

I certify that the attached documentation of Standard Quality Assurance Procedures and/or Standard Calibration Check Procedure are full, true and correct copy of the original.

Josh Sondrol 187  
Signature

STATE OF ARIZONA        )  
  ) ss  
COUNTY OF MOHAVE    )


Subscribed and sworn to before me this 27 day of November.

2019 by Josh Sondrol  
Carla Duarte

NOTARY PUBLIC

NOV. 27, 2019  
DATE

Commission expires:

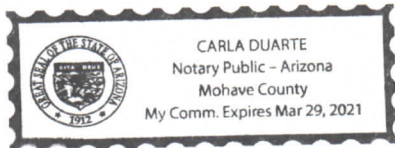

Area Code 928

Administration ..... 855-4884  
Business Office ..... 855-1171  
Investigations ..... 855-5775  
Patrol ..... 855-0515

Fax #'s  
Administration ..... 680-5430  
Business Office ..... 680-5431  
Investigations/Patrol ..... 680-5432

Emergency ..... 9-1-1  
Non-Emergency Dispatch/ ..... 855-4111  
Havasu Silent Witness ..... 854-TIPS

E-mail: police@lhcaz.gov

Exhibit G-2. Standard Calibration Check Procedure, Intoxilyzer Model 8000

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)  
ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J SONDRA 187 AGENCY LNC PD  
DATE 11/14/19 TIME 0914  
INTOXILYZER SERIAL # 80-003355 LOCATION JAIL

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34° C ± 0.2° C
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 102 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] 187

DPS Form Exh G-2 (Rev 05-01)

**Historical Note**  
New Exhibit G-2 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

\* THIS IS AN AMENDED G-2 FORM SHOWING THE CAL CHECK OF  
0.102 THAT WAS LEFT OFF ON THE ORIGINAL.

INTOXILYZER 8000  
Location: JAIL  
Serial Number: 80-003355  
Core Version: 8105.48  
11/14/2019 09:16:12

-----  
Standard Lot#: 13518100A1  
Last Changed By: J. SONDRÖL #187  
-----

QAS: J. SONDRÖL #187  
LAKE HAVASU CITY PD  
-----

Operator: J. SONDRÖL #187  
LAKE HAVASU CITY PD  
-----

Test	g/210L	Time
Air Blank	0.000	09:16:41
0.100 Cal Check	0.102	09:17:02
Air Blank	0.000	09:17:32

Preventative Maintenance  
Performed -- Timer Reset