

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)
ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. MALOY #281 AGENCY LHCPS
DATE 11-19-2020 TIME 0604
INTOXILYZER SERIAL # 80-003356 LOCATION SIGN 8

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 099 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Michael Maloy Jr #281

281

INTOXILYZER 8000
Location: CHANNEL SIGN 8
Serial Number: 80-003356
Core Version: 8105.48
11/19/2020 01:04:03

Standard Lot#: 35019100A1
Last Changed By: M. MALOY JR #281

QAS: M. MALOY JR #281
LAKE HAVASU CITY PD

Operator: M. MALOY JR #281
LAKE HAVASU CITY PD

Test	g/210L	Time
Air Blank	0.000	01:04:41
0.100 Cal Check	0.099	01:05:02
Air Blank	0.000	01:05:32

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST M. MALOY #281 AGENCY LHCPS
DATE 11-19-2020 TIME 0106
INTOXILYZER SERIAL # 80-003356 LOCATION SIGNB

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
- () 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
- () 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- () 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- () 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Michael Maloy Jr #281

281

INTOXILYZER 8000
Location: CHANNEL SIGN 8
Serial Number: 80-003356
Core Version: 8105.48
11/19/2020

01:06:40

Standard Lot#: 35019100A1
Last Changed By: M. MALOY JR #281

QAS: M. MALOY JR #281
LAKE HAVASU CITY PD

Operator: M. MALOY JR #281
LAKE HAVASU CITY PD

Subject: SQAP,TEST
DOB: 01/01/2000
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	01:07:39
Diagnostic Test	Pass	01:08:14
Air Blank	0.000	01:08:41
0.100 Cal Check	0.100	01:09:02
Air Blank	0.000	01:09:32
Subject Test	0.000	01:09:52
Air Blank	ABT*	01:09:59
Air Blank	IPS**	01:10:07

*Sequence Aborted
**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
 Location: CHANNEL SIGN 8
 Serial Number: 80-003356
 Core Version: 8105.48
 11/19/2020

01:12:17

Standard Lot#: 35019100A1
 Last Changed By: M. MALOY JR #281

QAS: M. MALOY JR #281
 LAKE HAVASU CITY PD

Operator: M. MALOY JR #281
 LAKE HAVASU CITY PD

Subject: SQAP, TEST
 DOB: 01/01/2000
 Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	01:13:09
Diagnostic Test	Pass	01:13:44
Air Blank	0.000	01:14:11
0.100 Cal Check	0.100	01:14:32
Air Blank	0.000	01:15:01
Subject Test	0.000*	01:18:09
Air Blank	0.000	01:18:36
Air Blank	0.000	01:19:04
Subject Test	XXX**	01:19:28
Air Blank	RFI***	01:19:37

*Deficient Sample

**Invalid Sample

Begin new deprivation period

***RFI Detect

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. MALDY #281 AGENCY LHCPO

DATE 11-19-2020 TIME 0121

INTOXILYZER SERIAL # 80-003356 LOCATION SIGN B

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 098 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Michael Malby Jr #281

281

INTOXILYZER 8000
Location: CHANNEL SIGN 8
Serial Number: 80-003356
Core Version: 8105.48
11/19/2020 01:21:46

Standard Lot#: 35019100A1
Last Changed By: M. MALOY JR #281

QAS: M. MALOY JR #281
LAKE HAVASU CITY PD

Operator: M. MALOY JR #281
LAKE HAVASU CITY PD

Test	g/210L	Time
Air Blank	0.000	01:22:16
0.100 Cal Check	0.098	01:22:37
Air Blank	0.000	01:23:07

Preventative Maintenance
Performed -- Timer Reset