

Exhibit G-2. Standard Calibration Check Procedure, Intoxilyzer Model 8000

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)
ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. Sanchez 187 AGENCY LNC PD
DATE 11/20/18 TIME 1204
INTOXILYZER SERIAL # 80-003355 LOCATION JAIL

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34° C ± 0.2° C
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 10.7 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

DPS Form Exh G-2 (Rev 05-01)

Historical Note
New Exhibit G-2 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

INTOXILYZER 8000
Location: JAIL
Serial Number: 80-003355
Core Version: 8105.48
11/20/2018

12:04:11

Standard Lot#: 13917100A1
Last Changed By: J. SONDRÖL #187

QAS: J. SONDRÖL #187
LAKE HAVASU CITY PD

Operator: J. SONDRÖL #187
LAKE HAVASU CITY PD

Test	g/210L	Time
Air Blank	0.000	12:04:42
0.100 Cal Check	0.101	12:05:03
Air Blank	0.000	12:05:32

Preventative Maintenance
Performed -- Timer Reset

J181