



Exhibit G-2. Standard Calibration Check Procedure, Intoxilyzer Model 8000

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. REG. 10.164(A)  
ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST LOUIS M. PROBERT III AGENCY MACPD

DATE 0-16-2019 TIME 0740

INTOXILYZER SERIAL # 80-003355 LOCATION MACPD JAIL

(X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR

Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34° C ± 0.2° C

- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0.099 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

DPS Form Exh G-2 (Rev 05-01)

**Historical Note**  
New Exhibit G-2 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

INTOXILYZER 8000

Location: JAIL

Serial Number: 80-003355

Core Version: 8105.48

10/16/2019

07:40:17

-----  
Standard Lot#: 13518100A1

Last Changed By: J. SONDROL #187  
-----

QAS: L. PLUNKETT III #253

LAKE HAVASU CITY PD  
-----

Operator: L. PLUNKETT III #253

LAKE HAVASU CITY PD  
-----

Test	g/310L	Time
Air Blank	0.000	07:41:14
0.100 Cal Check	0.099	07:41:35
Air Blank	0.000	07:42:05

Preventative Maintenance  
Performed - Timer Reset