



POLICE DEPARTMENT  
 LAKE HAVASU CITY  
 2360 McCULLOCH BOULEVARD NORTH  
 LAKE HAVASU CITY, ARIZONA 86403-5947

To Whom It May Concern:

I hereby attest that Josh Sondrol is a Certified Intoxilyzer Quality Assurance Specialist for the Lake Havasu City Police Department.

I certify that the attached documentation of Standard Quality Assurance Procedures and/or Standard Calibration Check Procedure are full, true and correct copy of the original.

*Josh Sondrol*  
 Signature

STATE OF ARIZONA        )  
   ) ss  
 COUNTY OF MOHAVE     )


Subscribed and sworn to before me this 19 day of October.

2018 by JOSH SONDROL

*Carla Duarte*  
 NOTARY PUBLIC

10-19-18  
 DATE

Commission expires:

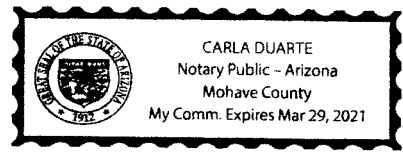

Area Code 928

Administration .....	855-4884	Fax #'s		Emergency .....	9-1-1
Business Office .....	855-1171	Administration .....	680-5430	Non-Emergency Dispatch/ .....	855-4111
Investigations .....	855-5775	Business Office .....	680-5431	Havasu Silent Witness .....	854-TIPS
Patrol .....	855-0515	Investigations/Patrol .....	680-5432		

E-mail: police@lhcaz.gov

Exhibit G-2. Standard Calibration Check Procedure, Intoxilyzer Model 8000

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)  
ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J SANDRUC 187 AGENCY LNC PD  
DATE 10/19/13 TIME 0608  
INTOXILYZER SERIAL # 80-003355 LOCATION JAIL

- (  ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution \_\_\_\_\_ AC.  
OR  
Pour a standard alcohol concentration solution .100 AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34° C ± 0.2° C
- (  ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (  ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (  ) 4. Air blank completed.
- (  ) 5. Calibration check completed. Test results 0. 100 AC.
- (  ) 6. Air blank completed.
- (  ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J Sandruc 187

DPS Form Exh G-2 (Rev 05-01)

Historical Note

New Exhibit G-2 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

Exhibit G-4. Standard Quality Assurance Procedure Intoxilyzer, Model 8000

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST J. Sandoval 187 AGENCY LVC AB  
DATE 10/19/18 TIME 0611  
INTOXILYZER SERIAL # 80-003355 LOCATION JAIL

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
- () 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
- () 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- () 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- () 4. Standard Calibration Check standard 0. 100 AC. Result 0. 101 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS SHOWED OUT OF TOLERANCE. CHANGED TANK + DID  
G-2, G-4, G-2. OPERATING NORMALLY.

SIGNATURE J. Sandoval 187

DPS Form Exh G-4 (Rev 05-01)

Historical Note

New Exhibit G-4 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

INTOXILYZER 8000  
Location: JAIL  
Serial Number: 80-003355  
Core Version: 8105.48  
10/19/2018

06:07:48

---

Standard Lot#: 13917100A1  
Last Changed By: J. SONDROL #187

---

QAS: J. SONDROL #187  
LAKE HAVASU CITY PD

---

Operator: J. SONDROL #187  
LHC PD

---

Subject: SQAP,TEST  
DOB: 10/05/1979  
Sex: M Weight: 123

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	06:08:49
Diagnostic Test	Pass	06:09:24
Air Blank	0.000	06:09:52
0.100 Cal Check	0.101	06:10:12
Air Blank	0.000	06:10:42
Subject Test	0.000	06:11:05
Air Blank	ABT*	06:11:11
Air Blank	IPS**	06:11:19

\*Sequence Aborted

\*\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: JAIL  
Serial Number: 80-003355  
Core Version: 8105.48  
10/19/2018

06:11:38

---

Standard Lot#: 13917100A1  
Last Changed By: J. SONDROL #187

---

QAS: J. SONDROL #187  
LAKE HAVASU CITY PD

---

Operator: J. SONDROL #187  
LAKE HAVASU CITY PD

---

Subject: SQAP,TEST  
DOB: 10/05/1979  
Sex: M Weight: 123

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	06:12:31
Diagnostic Test	Pass	06:13:06
Air Blank	0.000	06:13:33
0.100 Cal Check	0.101	06:13:54
Air Blank	0.000	06:14:24
Subject Test	0.165*	06:17:31
Air Blank	0.000	06:18:05
Air Blank	0.000	06:18:33
Subject Test	XXX**	06:19:03
Air Blank	RFI***	06:19:13

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

\*\*\*RFI Detect

**Not a Successfully Completed Test Sequence**

Exhibit G-2. Standard Calibration Check Procedure, Intoxilyzer Model 8000

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)  
ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. SANDER 187 AGENCY LNC PD  
DATE 12/19/18 TIME 0650  
INTOXILYZER SERIAL # 80-003355 LOCATION JW12

- (  ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34° C ± 0.2° C
- (  ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (  ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (  ) 4. Air blank completed.
- (  ) 5. Calibration check completed. Test results 0. 102 AC.
- (  ) 6. Air blank completed.
- (  ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

DPS Form Exh G-2 (Rev 05-01)

**Historical Note**  
New Exhibit G-2 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

INTOXILYZER 8000

Location: JAIL

Serial Number: 80-003355

Core Version: 8105.48

10/19/2018

06:20:12

-----  
Standard Lot#: 13917100A1

Last Changed By: J. SONDROL #187  
-----

QAS: J. SONDROL #187

LAKE HAVASU CITY PD  
-----

Operator: J. SONDROL #187

LAKE HAVASU CITY PD  
-----

Test	g/210L	Time
Air Blank	0.000	06:20:41
0.100 Cal Check	0.102	06:21:02
Air Blank	0.000	06:21:32

Preventative Maintenance  
Performed -- Timer Reset