



POLICE DEPARTMENT  
**LAKE HAVASU CITY**  
 2360 McCULLOCH BOULEVARD NORTH  
 LAKE HAVASU CITY, ARIZONA 86403-5947

To Whom It May Concern:

I hereby attest that Michael T. Maloy, Jr. is a Certified Intoxilyzer Quality Assurance Specialist for the Lake Havasu City Police Department.

I certify that the attached documentation of Standard Quality Assurance Procedures and/or Standard Calibration Check Procedure are full, true and correct copy of the original.

Michael Maloy Jr #281  
 Signature

STATE OF ARIZONA        )  
   ) ss  
 COUNTY OF MOHAVE     )


Subscribed and sworn to before me this 15 day of September.

2021 by Ashley Rae Bonds

Ashley Rae Bonds  
 NOTARY PUBLIC

9-15-21  
 DATE

Commission expires: May 29, 2025

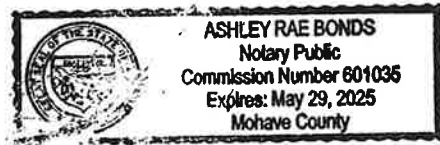

Area Code 928

Administration ..... 855-4884  
 Business Office ..... 855-1171  
 Investigations ..... 855-5775  
 Patrol ..... 855-0515

Fax #'s  
 Administration ..... 680-5430  
 Business Office ..... 680-5431  
 Investigations/Patrol ..... 680-5432

Emergency ..... 9-1-1  
 Non-Emergency Dispatch/ ..... 855-4111  
 Havasu Silent Witness ..... 854-TIPS

E-mail: police@lhcaz.gov

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. MALOY #281 AGENCY LNCPD  
DATE 09-15-2021 TIME 1554  
INTOXILYZER SERIAL # 80-001029 LOCATION LNCPD JAIL

- (  ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (  ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (  ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (  ) 4. Air blank completed.
- (  ) 5. Calibration check completed. Test results 0. 096 AC.
- (  ) 6. Air blank completed.
- (  ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Michael Maloy #281

281

INTOXILYZER 8000  
Location: LHCPD  
Serial Number: 80-001029  
Core Version: 8105.48  
09/15/2021 15:54:34

-----  
Standard Lot#: 35019100A1  
Last Changed By: K. TREBES #7259  
-----

QAS: M. MALOY JR #281  
LAKE HAVASU CITY PD  
-----

Operator: M. MALOY JR #281  
LAKE HAVASU CITY PD  
-----

Test	g/210L	Time
Air Blank	0.000	15:55:04
0.100 Cal Check	0.096	15:55:24
Air Blank	0.000	15:55:54

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST M. MALOY #281 AGENCY LHCPD  
DATE 09-15-2021 TIME 1557  
INTOXILYZER SERIAL # 80-001029 LOCATION LHCPD JAIL

(  ) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (  ) 1. Clock time check.  
(  ) 2. Date check.

OPERATIONAL TESTS

- (  ) 1. Alcohol-free subject test result 0. 000 AC.  
(  ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
(  ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
(  ) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE Michael Maloy #281

281

INTOXILYZER 8000  
Location: LHCPD  
Serial Number: 80-001029  
Core Version: 8105.48  
09/15/2021 15:57:12

---

Standard Lot#: 35019100A1  
Last Changed By: K. TREBES #7259

---

QAS: M. MALOY JR #281  
LAKE HAVASU CITY PD

---

Operator: M. MALOY JR #281  
LAKE HAVASU CITY PD

---

Subject: SQAP,TEST  
DOB: 01/01/2000  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	15:58:03
Diagnostic Test	Pass	15:58:38
Air Blank	0.000	15:59:06
0.100 Cal Check	0.098	15:59:26
Air Blank	0.000	15:59:56
Subject Test	0.000	16:00:18
Air Blank	ABT*	16:00:25
Air Blank	IPS**	16:00:33

\*Sequence Aborted  
\*\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: LHCPD  
Serial Number: 80-001029  
Core Version: 8105.48  
09/15/2021 16:01:15

---

Standard Lot#: 35019100A1  
Last Changed By: K. TREBES #7259

---

QAS: M. MALOY JR #281  
LAKE HAVASU CITY PD

---

Operator: M. MALOY JR #281  
LAKE HAVASU CITY PD

---

Subject: SQAP,TEST  
DOB: 01/01/2000  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	16:02:05
Diagnostic Test	Pass	16:02:41
Air Blank	0.000	16:03:08
0.100 Cal Check	0.098	16:03:29
Air Blank	0.000	16:03:58
Subject Test	0.000*	16:07:06
Air Blank	0.000	16:07:33
Air Blank	0.000	16:08:01
Subject Test	XXX**	16:08:15
Air Blank	RFI***	16:08:26

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

\*\*\*RFI Detect

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. MALOY #281 AGENCY LKCPD  
DATE 09-15-2021 TIME 1609  
INTOXILYZER SERIAL # 80-001029 LOCATION LKCPD JAIL

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 097 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Michael Maloy #281

281

INTOXILYZER 8000  
Location: LHCPD  
Serial Number: 80-001029  
Core Version: 8105.48  
09/15/2021 16:09:11

-----  
Standard Lot#: 35019100A1  
Last Changed By: K. TREBES #7259  
-----

QAS: M. MALOY JR #281  
LAKE HAVASU CITY PD  
-----

Operator: M. MALOY JR #281  
LAKE HAVASU CITY PD  
-----

Test	g/210L	Time
Air Blank	0.000	16:09:40
0.100 Cal Check	0.097	16:10:01
Air Blank	0.000	16:10:31

Preventative Maintenance  
Performed -- Timer Reset