



POLICE DEPARTMENT  
**LAKE HAVASU CITY**

2360 McCULLOCH BOULEVARD NORTH  
LAKE HAVASU CITY, ARIZONA 86403-5947

To Whom It May Concern:

I hereby attest that Michael T. Maloy, Jr. is a Certified Intoxilyzer Quality Assurance Specialist for the Lake Havasu City Police Department.

I certify that the attached documentation of Standard Quality Assurance Procedures and/or Standard Calibration Check Procedure are full, true and correct copy of the original.

Michael Maloy Jr  
Signature

STATE OF ARIZONA        )  
  ) ss  
COUNTY OF MOHAVE    )

Subscribed and sworn to before me this 29 day of June.

2020 by Ashley Rae Bonds

Ashley Rae Bonds  
NOTARY PUBLIC

6-29-2020  
DATE

Commission expires: May 29, 2021



Area Code 928

Administration ..... 855-4884  
Business Office ..... 855-1171  
Investigations ..... 855-5775  
Patrol ..... 855-0515

Fax #'s  
Administration ..... 680-5430  
Business Office ..... 680-5431  
Investigations/Patrol ..... 680-5432

Emergency ..... 9-1-1  
Non-Emergency Dispatch/ ..... 855-4111  
Havasu Silent Witness ..... 854-TIPS

E-mail: police@lhcaz.gov

281

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. MALOY #281 AGENCY LUCPS

DATE 06-30-2020 TIME 0038

INTOXILYZER SERIAL # 80-001029 LOCATION MOBILE BOOKING STATION

- (  ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (  ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (  ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (  ) 4. Air blank completed.
- (  ) 5. Calibration check completed. Test results 0. 099 AC.
- (  ) 6. Air blank completed.
- (  ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Michael Maloy Jr

281

INTOXILYZER 8000  
Instrument Initialization  
19:54:58 06/29/2020

INTOXILYZER 8000  
Location: CHANNEL BOOKING  
Serial Number: 80-001029  
Core Version: 8105.48  
06/30/2020 00:38:07

-----  
Standard Lot#: 35019100A1  
Last Changed By: M. MALOY JR #281  
-----

QAS: M. MALOY JR #281  
LAKE HAVASU CITY PD  
-----

Operator: M. MALOY JR #281  
LAKE HAVASU CITY PD  
-----

Test	g/210L	Time
Air Blank	0.000	00:38:39
0.100 Cal Check	0.099	00:39:00
Air Blank	0.000	00:39:29

Preventative Maintenance  
Performed -- Timer Reset

281

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST M. MALOY #281 AGENCY LUCPD

DATE 06-30-2020 TIME 0041

INTOXILYZER SERIAL # 80-001029 LOCATION MOBILE BOOKING STATION

(  ) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (  ) 1. Clock time check.
- (  ) 2. Date check.

OPERATIONAL TESTS


- (  ) 1. Alcohol-free subject test result 0. 000 AC.
- (  ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- (  ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- (  ) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

PLACED IN SERVICE FOR 4TH OF JULY  
WEEKEND

SIGNATURE

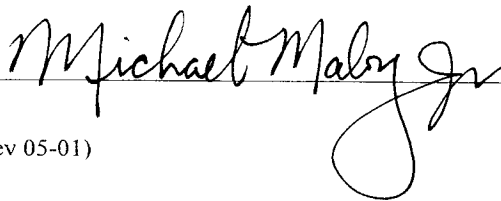

281

INTOXILYZER 8000  
Location: CHANNEL BOOKING  
Serial Number: 80-001029  
Core Version: 8105.48  
06/30/2020 00:41:36

Standard Lot#: 35019100A1  
Last Changed By: M. MALOY JR #281

OAS: M. MALOY JR #281  
LAKE HAVASU CITY PD

Operator: M. MALOY JR #281  
LAKE HAVASU CITY PD

Subject: SOAP,TEST  
DOB: 01/01/2000  
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	00:42:45
Diagnostic Test	Pass	00:43:23
Air Blank	0.000	00:43:50
0.100 Cal Check	0.099	00:44:11
Air Blank	0.000	00:44:40
Subject Test	0.000	00:45:00
Air Blank	ABT*	00:45:06
Air Blank	IPS**	00:45:15

\*Sequence Aborted  
\*\*Improper Sample

Not a Successfully  
Completed  
Test Sequence

281

INTOXILYZER 8000  
Location: CHANNEL BOOKING  
Serial Number: 80-001029  
Core Version: 8105.48  
06/30/2020 00:45:50

Standard Lot#: 35019100A1  
Last Changed By: M. MALOY JR #281

OAS: M. MALOY JR #281  
LAKE HAVASU CITY PD

Operator: M. MALOY JR #281  
LAKE HAVASU CITY PD

Subject: SOAP,TEST  
DOB: 01/01/2000  
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	00:46:40
Diagnostic Test	Pass	00:47:18
Air Blank	0.000	00:47:45
0.100 Cal Check	0.100	00:48:06
Air Blank	0.000	00:48:35
Subject Test	0.000*	00:51:43
Air Blank	0.000	00:52:10
Air Blank	0.000	00:52:37
Subject Test	XXX**	00:52:48
Air Blank	RFI***	00:52:57

\*Deficient Sample  
\*\*Invalid Sample  
Begin new deprivation period  
\*\*\*RFI Detect

Not a Successfully  
Completed  
Test Sequence

281

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)  
ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. MALOY #281 AGENCY LUCPD

DATE 06-30-2020 TIME 0053

INTOXILYZER SERIAL # 80-001029 LOCATION MOBILE BOOKING STATION

- (  ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (  ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (  ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (  ) 4. Air blank completed.
- (  ) 5. Calibration check completed. Test results 0. 099 AC.
- (  ) 6. Air blank completed.
- (  ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Michael Maloy Jr

281

INTOXILYZER 8000  
Location: CHANNEL BOOKING  
Serial Number: 80-001029  
Core Version: 8105.48  
06/30/2020 00:53:40

-----  
Standard Lot#: 35019:00A1  
Last Changed By: M. MALOY JR #281  
-----

DAS: M. MALOY JR #281  
LAKE HAVASU CITY PD  
-----

Operator: M. MALOY JR #281  
LAKE HAVASU CITY PD  
-----

Test	g/210L	Time
Air Blank	0.000	00:54:11
0.100 Cal Check	0.099	00:54:31
Air Blank	0.000	00:55:01

Preventative Maintenance  
Performed -- Timer Reset