

Exhibit G-2. Standard Calibration Check Procedure, Intoxilyzer Model 8000

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)


ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST LOUIS H. PLUNKETT III 253 AGENCY LHCPDDATE 04/17/18 TIME 0753INTOXILYZER SERIAL # 80-001029 LOCATION LHCPD JAIL

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34° C ± 0.2° C

2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0. .100 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE  253

DPS Form Exh G-2 (Rev 05-01)

Historical Note

New Exhibit G-2 made by final rulcmaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

INTOXILYZER 8000
Location: MAIN JAIL
Serial Number: 80-001029
Core Version: 8105.48
04/17/2018

07:53:57

Standard Lot#: 13917100A1
Last Changed By: J. SONDRON #187

QAS: L. PLUNKETT III #253
LAKE HAVASU CITY PD

Operator: L. PLUNKETT III #253
LAKE HAVASU CITY PD

Test	g/210L	Time
Air Blank	0.000	07:54:28
0.100 Cal Check	0.100	07:54:49
Air Blank	0.000	07:55:19

Preventative Maintenance
Performed -- Timer Reset

Exhibit G-4. Standard Quality Assurance Procedure Intoxilyzer, Model 8000

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST LOUIS H. PLUNKETT III 253 AGENCY LHCPD

DATE 04/17/18 TIME 0758

INTOXILYZER SERIAL # 80-001029 LOCATION LHCPD JAIL

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS


- 1. Clock time check.
- 2. Date check.

OPERATIONAL TESTS

- 1. Alcohol-free subject test result 0. 000 AC.
- 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- 4. Standard Calibration Check standard 0. 100 AC. Result 0. 100 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS _____

SIGNATURE  253

DPS Form Exh G-4 (Rev 05-01)

Historical Note

New Exhibit G-4 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

INTOXILYZER 8000
Location: MAIN JAIL
Serial Number: 80-001029
Core Version: 8105.48
04/17/2018

07:58:21

Standard Lot#: 13917100A1
Last Changed By: J. SONDROL #187

QAS: L. PLUNKETT III #253
LAKE HAVASU CITY PD

Operator: L. PLUNKETT III #253
LAKE HAVASU CITY PD

Subject: SQAP,TEST
DOB: 01/01/2000
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	07:59:33
Diagnostic Test	Pass	08:00:08
Air Blank	0.000	08:00:35
0.100 Cal Check	0.100	08:00:56
Air Blank	0.000	08:01:26
Subject Test	0.000	08:01:44
Air Blank	ABT*	08:01:50
Air Blank	IPS**	08:01:58

*Sequence Aborted

**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: MAIN JAIL
Serial Number: 80-001029
Core Version: 8105.48
04/17/2018

08:02:49

Standard Lot#: 13917100A1
Last Changed By: J. SONDROL #187

QAS: L. PLUNKETT III #253
LAKE HAVASU CITY PD

Operator: L. PLUNKETT III #253
LAKE HAVASU CITY PD

Subject: SQAP,TEST
DOB: 01/01/2000
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:03:43
Diagnostic Test	Pass	08:04:18
Air Blank	0.000	08:04:45
0.100 Cal Check	0.100	08:05:06
Air Blank	0.000	08:05:35
Subject Test	0.102*	08:08:43
Air Blank	0.000	08:09:15
Air Blank	0.000	08:09:42
Subject Test	XXX**	08:10:17
Air Blank	RFI***	08:10:26

*Deficient Sample

**Invalid Sample

Begin new deprivation period

***RFI Detect

Not a Successfully Completed Test Sequence

Exhibit G-2. Standard Calibration Check Procedure, Intoxilyzer Model 8000

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)
ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST LOUIS H. PLUNKETT JR 263 AGENCY LHCPD

DATE 04/17/18 TIME 0814

INTOXILYZER SERIAL # 80-001029 LOCATION LHCPD JAIL

(X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR

Pour a standard alcohol concentration solution _____AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34° C ± 0.2° C

(X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"


(X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.

(X) 4. Air blank completed.

(X) 5. Calibration check completed. Test results 0.099 AC.

(X) 6. Air blank completed.

(X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE  263

DPS Form Exh G-2 (Rev 05-01)

Historical Note

New Exhibit G-2 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

INTOXILYZER 8000
Location: MAIN JAIL
Serial Number: 80-001029
Core Version: 8105.48
04/17/2018

08:14:25

Standard Lot#: 13917100A1
Last Changed By: J. SONDRON #187

QAS: L. PLUNKETT III #253
LAKE HAVASU CITY PD

Operator: L. PLUNKETT III #253
LAKE HAVASU CITY PD

Test	g/210L	Time
Air Blank	0.000	08:14:56
0.100 Cal Check	0.099	08:15:17
Air Blank	0.000	08:15:46

Preventative Maintenance
Performed -- Timer Reset