



POLICE DEPARTMENT
LAKE HAVASU CITY
 2360 McCULLOCH BOULEVARD NORTH
 LAKE HAVASU CITY, ARIZONA 86403-5947

To Whom It May Concern:

I hereby attest that Louis H. Plunkett III is a Certified Intoxilyzer Quality Assurance Specialist for the Lake Havasu City Police Department.

I certify that the attached documentation of Standard Quality Assurance Procedures and/or Standard Calibration Check Procedure are full, true and correct copy of the original.




 Signature

STATE OF ARIZONA)
) ss
 COUNTY OF MOHAVE)

Subscribed and sworn to before me this 19th day of FEBRUARY.

2018 by WANDA R. SKINNER




 NOTARY PUBLIC

Feb. 19, 2018

 DATE

Commission expires:
Feb. 24, 2021

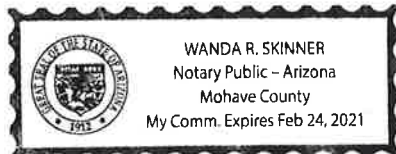

Area Code 928

Administration 855-4884
 Business Office 855-1171
 Investigations 855-5775
 Patrol 855-0515

Fax #'s
 Administration 680-5430
 Business Office 680-5431
 Investigations/Patrol 680-5432

Emergency 9-1-1
 Non-Emergency Dispatch/ 855-4111
 Havasu Silent Witness 854-TIPS

E-mail: police@lhcaz.gov

Exhibit G-2. Standard Calibration Check Procedure, Intoxilyzer Model 8000

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)
ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST LOUIS H. PHUNKETT III 253 AGENCY LHCPD

DATE 02/19/18 TIME 0830

INTOXILYZER SERIAL # 80-001029 LOCATION LHCPD JAIL

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34° C ± 0.2° C
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. 100 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE  253

DPS Form Exh G-2 (Rev 05-01)

Historical Note
New Exhibit G-2 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

INTOXILYZER 8000
Location: MAIN JAIL
Serial Number: 80-001029
Core Version: 8105.48
02/19/2018 08:30:00

Standard Lot#: 13917100A1
Last Changed By: J. SONDRON #187

QAS: L. PLUNKETT III #253
LAKE HAVASU CITY PD

Operator: L. PLUNKETT III #253
LAKE HAVASU CITY PD

Test	g/210L	Time
Air Blank	0.000	08:30:33
0.100 Cal Check	0.100	08:30:54
Air Blank	0.000	08:31:24

Preventative Maintenance
Performed -- Timer Reset