



POLICE DEPARTMENT

2360 McCULLOCH BOULEVARD NORTH
LAKE HAVASU CITY, ARIZONA 86403-5947

To Whom It May Concern:

I hereby attest that Josh Sondrol is a Certified Intoxilyzer Quality Assurance Specialist for the Lake Havasu City Police Department.

I certify that the attached documentation of Standard Quality Assurance Procedures and/or Standard Calibration Check Procedure are full, true and correct copy of the original.

Josh Sondrol 157
Signature

STATE OF ARIZONA)
) ss
COUNTY OF MOHAVE)


Subscribed and sworn to before me this 21ST day of JANUARY.

248 by Josh Sondrol

Shawna Mcknight
NOTARY PUBLIC

1/21/18
DATE

Commission expires: MAY 4, 2021

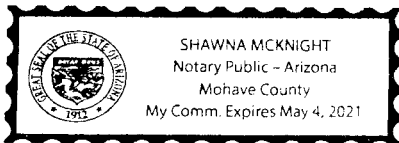

Area Code 928

Administration 855-4884
Business Office 855-1171
Investigations 855-5775
Patrol 855-0515

Fax #'s
Administration 680-5430
Business Office 680-5431
Investigations/Patrol 680-5432

Emergency 9-1-1
Non-Emergency Dispatch/ 855-4111
Havasu Silent Witness 854-TIPS

E-mail: police@lhcaz.gov

Exhibit G-2. Standard Calibration Check Procedure, Intoxilyzer Model 8000

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)
ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J SONDRAL 187 AGENCY LNC PD

DATE 1/21/18 TIME 1542

INTOXILYZER SERIAL # 80-001029 LOCATION MAIN JAIL

() 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34° C ± 0.2° C

- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 101 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J Sondral 187

DPS Form Exh G-2 (Rev 05-01)

Historical Note

New Exhibit G-2 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

4187

INTOXILYZER 8000
Location: MAIN JAIL
Serial Number: 80-001029
Core Version: 8105.48
01/21/2018 15:42:06

Standard Lot#: 13917100A1
Last Changed By: J. SONDROL #187

QAS: J. SONDROL #187
LAKE HAVASU CITY PD

Operator: J. SONDROL #187
LAKE HAVASU CITY PD

Test	g/210L	Time
Air Blank	0.000	15:42:46
0.100 Cal Check	0.101	15:43:06
Air Blank	0.000	15:43:36

Preventative Maintenance
Performed -- Timer Reset

Exhibit G-4. Standard Quality Assurance Procedure Intoxilyzer, Model 8000

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST J Sondroc 187 AGENCY LNC PD

DATE 1/21/18 TIME 1545

INTOXILYZER SERIAL # 80-001029 LOCATION MAIN JAIL

(✓ 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

(✓ 1. Clock time check.

(✓ 2. Date check.

OPERATIONAL TESTS

(✓ 1. Alcohol-free subject test result 0. 000 AC.

(✓ 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed

(✓ 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed

(✓ 4. Standard Calibration Check standard 0. 100 AC. Result 0. 101 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS ONE DAY (30) BEHIND

SIGNATURE J Sondroc 187

DPS Form Exh G-4 (Rev 05-01)

Historical Note

New Exhibit G-4 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

98187

INTOXILYZER 8000
Location: MAIN JAIL
Serial Number: 80-001029
Core Version: 8105.48
01/21/2018

15:45:07

Standard Lot#: 13917100A1
Last Changed By: J. SONDROL #187

QAS: J. SONDROL #187
LAKE HAVASU CITY PD

Operator: J. SONDROL #187
LAKE HAVASU CITY PD

Subject: SQAP,TEST
DOB: 01/01/1950
Sex: M Weight: 123

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:45:55
Diagnostic Test	Pass	15:46:30
Air Blank	0.000	15:46:57
0.100 Cal Check	0.101	15:47:18
Air Blank	0.000	15:47:48
Subject Test	0.000	15:48:06
Air Blank	ABT*	15:48:12
Air Blank	IPS**	15:48:20

*Sequence Aborted
**Improper Sample

Not a Successfully Completed Test Sequence

2187

INTOXILYZER 8000
Location: MAIN JAIL
Serial Number: 80-001029
Core Version: 8105.48
01/21/2018 15:48:49

Standard Lot#: 13917100A1
Last Changed By: J. SONDROL #187

QAS: J. SONDROL #187
LAKE HAVASU CITY PD

Operator: J. SONDROL #187
LAKE HAVASU CITY PD

Subject: SQAP,TEST
DOB: 01/10/1950
Sex: M Weight: 123

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:49:34
Diagnostic Test	Pass	15:50:09
Air Blank	0.000	15:50:36
0.100 Cal Check	0.101	15:50:57
Air Blank	0.000	15:51:27
Subject Test	0.106*	15:54:34
Air Blank	0.000	15:55:07
Air Blank	0.000	15:55:34
Subject Test	XXX**	15:56:01
Air Blank	RFI***	15:56:09

*Deficient Sample
**Invalid Sample
 Begin new deprivation period
***RFI Detect

Not a Successfully Completed Test Sequence

Exhibit G-2. Standard Calibration Check Procedure, Intoxilyzer Model 8000

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)
ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J SANDRONE 187 AGENCY LHC PD
DATE 1/21/18 TIME 1556
INTOXILYZER SERIAL # 80-001029 LOCATION MAIN JAIL

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34° C ± 0.2° C
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 100 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J Sandrone 187

DPS Form Exh G-2 (Rev 05-01)

Historical Note

New Exhibit G-2 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

95187

INTOXILYZER 8000
Location: MAIN JAIL
Serial Number: 80-001029
Core Version: 8105.48
01/21/2018 15:56:55

Standard Lot#: 13917100A1
Last Changed By: J. SONDROL #187

QAS: J. SONDROL #187
LAKE HAVASU CITY PD

Operator: J. SONDROL #187
LAKE HAVASU CITY PD

Test	g/210L	Time
Air Blank	0.000	15:57:23
0.100 Cal Check	0.100	15:57:44
Air Blank	0.000	15:58:14

Preventative Maintenance
Performed -- Timer Reset