



POLICE DEPARTMENT  
**LAKE HAVASU CITY**  
 2360 McCULLOCH BOULEVARD NORTH  
 LAKE HAVASU CITY, ARIZONA 86403-5947

To Whom It May Concern:

I hereby attest that Louis H. Plunkett III is a Certified Intoxilyzer Quality Assurance Specialist for the Lake Havasu City Police Department.

I certify that the attached documentation of Standard Quality Assurance Procedures and/or Standard Calibration Check Procedure are full, true and correct copy of the original.

  
 \_\_\_\_\_  
 Signature

STATE OF ARIZONA            )  
                                           ) ss  
 COUNTY OF MOHAVE        )


Subscribed and sworn to before me this 15 day of March 2018

\_\_\_\_\_ by Ashley Bonds

  
 \_\_\_\_\_  
 NOTARY PUBLIC

3/15/18  
 \_\_\_\_\_  
 DATE

Commission expires: May 29, 2021

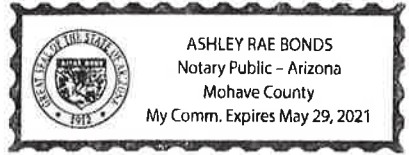

Area Code 928

Administration .....	855-4884	Fax #'s		Emergency .....	9-1-1
Business Office .....	855-1171	Administration .....	680-5430	Non-Emergency Dispatch/ .....	855-4111
Investigations .....	855-5775	Business Office .....	680-5431	Havasu Silent Witness .....	854-TIPS
Patrol .....	855-0515	Investigations/Patrol .....	680-5432		

E-mail: police@lhcaz.gov

Department of Public Safety - Alcohol Testing

Exhibit G-2. Standard Calibration Check Procedure, Intoxilyzer Model 8000

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)  
ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST LOUIS H. PLUNKETT III AGENCY LACPD

DATE 03/15/18 TIME 1556

INTOXILYZER SERIAL # 80-001029 LOCATION LACPD JAIL

(X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.

OR

Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34° C ± 0.2° C

(X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"

(X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.

(X) 4. Air blank completed.

(X) 5. Calibration check completed. Test results 0. .100 AC.

(X) 6. Air blank completed.

(X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE  253

DPS Form Exh G-2 (Rev 05-01)

Historical Note

New Exhibit G-2 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

INTOXILYZER 8000

Location: MAIN JAIL

Serial Number: 80-001029

Core Version: 8105.48

03/15/2018

15:56:13

-----  
Standard Lot#: 13917100A1

Last Changed By: J. SONDRON #187  
-----

QAS: L. PLUNKETT III #253

LAKE HAVASU CITY PD  
-----

Operator: L. PLUNKETT III #253

LAKE HAVASU CITY PD  
-----

Test	g/210L	Time
Air Blank	0.000	15:56:45
0.100 Cal Check	0.100	15:57:05
Air Blank	0.000	15:57:35

Preventative Maintenance  
Performed -- Timer Reset