



POLICE DEPARTMENT
LAKE HAVASU CITY
2360 McCULLOCH BOULEVARD NORTH
LAKE HAVASU CITY, ARIZONA 86403-5947

To Whom It May Concern:

I hereby attest that Josh Sondrol is a Certified Intoxilyzer Quality Assurance Specialist for the Lake Havasu City Police Department.

I certify that the attached documentation of Standard Quality Assurance Procedures and/or Standard Calibration Check Procedure are full, true and correct copy of the original.

[Handwritten Signature]
Signature

STATE OF ARIZONA)
) ss
COUNTY OF MOHAVE)

Subscribed and sworn to before me this 20 day of November.

[Handwritten Name] by _____

[Handwritten Signature]
NOTARY PUBLIC

11/20/17
DATE

Commission expires:



Area Code 928

Administration 855-4884
Business Office 855-1171
Investigations 855-5775
Patrol 855-0515

Fax #'s
Administration 680-5430
Business Office 680-5431
Investigations/Patrol 680-5432

Emergency 9-1-1
Non-Emergency Dispatch/ 855-4111
Havasu Silent Witness 854-TIPS

E-mail: police@lhcaz.gov

Exhibit G-2. Standard Calibration Check Procedure, Intoxilyzer Model 8000

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)
ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J SANDRZAL 187 AGENCY LNC PD
DATE 11/20/17 TIME 2156
INTOXILYZER SERIAL # 80-001029 LOCATION JR1L

- (1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34° C ± 0.2° C
- (2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (4. Air blank completed.
- (5. Calibration check completed. Test results 0. 100 AC.
- (6. Air blank completed.
- (7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J Sandrzal 187

DPS Form Exh G-2 (Rev 05-01)

Historical Note

New Exhibit G-2 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

INTOXILYZER 8000
Location: MAIN JAIL
Serial Number: 80-001029
Core Version: 8105.48
11/20/2017

21:59:26

Standard Lot#: 13917100A1
Last Changed By: J. SONDROL #187

QAS: J. SONDROL #187
LAKE HAVASU CITY PD

Operator: J. SONDROL #187
LAKE HAVASU CITY PD

Test	g/210L	Time
Air Blank	0.000	21:59:56
0.100 Cal Check	0.100	22:00:17
Air Blank	0.000	22:00:47

Preventative Maintenance
Performed -- Timer Reset

Exhibit G-4. Standard Quality Assurance Procedure Intoxilyzer, Model 8000

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST J SANDERL 187 AGENCY LUCAS
DATE 11/20/17 TIME 2203
INTOXILYZER SERIAL # 80-001029 LOCATION TRIL

(1. Display Reads "PUSH BUTTON TO START")

DIAGNOSTIC TESTS

- (1. Clock time check.
- (2. Date check.

OPERATIONAL TESTS

- (1. Alcohol-free subject test result 0. 000 AC.
- (2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (4. Standard Calibration Check standard 0. 100 AC. Result 0. 100 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS IN SERVICE B/C OTHER (3356) WENT FOR 1/4 MAIN

SIGNATURE J Sanderl 187

DPS Form Exh G-4 (Rev 05-01)

Historical Note

New Exhibit G-4 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

7107

INTOXILYZER 8000
Location: MAIN JAIL
Serial Number: 80-001029
Core Version: 8105.48
11/20/2017

22:03:14

Standard Lot#: 13917100A1
Last Changed By: J. SONDROL #187

QAS: J. SONDROL #187
LAKE HAVASU CITY PD

Operator: J. SONDROL #187
LAKE HAVASU CITY PD

Subject: SQAP,TEST
DOB: 01/01/1950
Sex: M Weight: 123

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	22:04:01
Diagnostic Test	Pass	22:04:36
Air Blank	0.000	22:05:04
0.100 Cal Check	0.100	22:05:25
Air Blank	0.000	22:05:54
Subject Test	0.000	22:06:14
Air Blank	ABT*	22:06:21
Air Blank	IPS**	22:06:28

*Sequence Aborted

**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: MAIN JAIL
Serial Number: 80-001029
Core Version: 8105.48
11/20/2017

22:07:15

Standard Lot#: 13917100A1
Last Changed By: J. SONDROL #187

QAS: J. SONDROL #187
LAKE HAVASU CITY PD

Operator: J. SONDROL #187
LAKE HAVASU CITY PD

Subject: SQAP,TEST
DOB: 01/01/1950
Sex: M Weight: 123

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	22:08:02
Diagnostic Test	Pass	22:08:37
Air Blank	0.000	22:09:04
0.100 Cal Check	0.100	22:09:25
Air Blank	0.000	22:09:55
Subject Test	0.363*	22:13:02
Air Blank	0.000	22:13:37
Air Blank	0.000	22:14:05
Subject Test	XXX**	22:14:31
Air Blank	RFI***	22:14:39

*Deficient Sample

**Invalid Sample

Begin new deprivation period

***RFI Detect

Not a Successfully Completed Test Sequence

Exhibit G-2. Standard Calibration Check Procedure, Intoxilyzer Model 8000

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)
ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. Sonora 187 AGENCY LNC PD
DATE 11/20/17 TIME 2216
INTOXILYZER SERIAL # 80-001029 LOCATION JAIL

(1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34° C ± 0.2° C

- (2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (4. Air blank completed.
- (5. Calibration check completed. Test results 0. 099 AC.
- (6. Air blank completed.
- (7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Sonora 187

DPS Form Exh G-2 (Rev 05-01)

Historical Note

New Exhibit G-2 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

INTOXILYZER 8000
Location: MAIN JAIL
Serial Number: 80-001029
Core Version: 8105.48
11/20/2017

22:16:35

Standard Lot#: 13917100A1
Last Changed By: J. SONDROL #187

QAS: J. SONDROL #187
LAKE HAVASU CITY PD

Operator: J. SONDROL #187
LAKE HAVASU CITY PD

Test	g/210L	Time
Air Blank	0.000	22:17:04
0.100 Cal Check	0.099	22:17:24
Air Blank	0.000	22:17:54

Preventative Maintenance
Performed -- Timer Reset