

281

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. MALDY #281 AGENCY LKCPD
DATE 07-19-2020 TIME 0711
INTOXILYZER SERIAL # 8D-001873 LOCATION LKCPD JAIL

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 098 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Michael Maloy #281

281

INTOXILYZER 8000
Location: LHCPD
Serial Number: 80-001873
Core Version: 8105.48
07/19/2020

07:11:59

Standard Lot#: 35019100A1
Last Changed By: M. MALOY JR #281

QAS: M. MALOY JR #281
LAKE HAVASU CITY PD

Operator: M. MALOY JR #281
LAKE HAVASU CITY PD

Test	g/210L	Time
Air Blank	0.000	07:12:31
0.100 Cal Check	0.098	07:12:51
Air Blank	0.000	07:13:21

Preventative Maintenance
Performed -- Timer Reset

281

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST M. MALDY #281 AGENCY LHCPD
DATE 07-19-2020 TIME 0715
INTOXILYZER SERIAL # 80-601873 LOCATION LHCPD JAIL

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
- () 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
- () 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- () 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- () 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

REMOVED BATTERY-BACKUP A.P.U.

SIGNATURE Michael Malady #281

281

INTOXILYZER 8000
Location: LHCPD
Serial Number: 80-001873
Core Version: 8105.48
07/19/2020

07:15:32

Standard Lot#: 35019100A1
Last Changed By: M. MALOY JR #281

QAS: M. MALOY JR #281
LAKE HAVASU CITY PD

Operator: M. MALOY JR #281
LAKE HAVASU CITY PD

Subject: SQAP, TEST
DOB: 01/01/2000
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	07:16:45
Diagnostic Test	Pass	07:17:20
Air Blank	0.000	07:17:48
0.100 Cal Check	0.099	07:18:08
Air Blank	0.000	07:18:38
Subject Test	0.000	07:19:03
Air Blank	ABT*	07:19:10
Air Blank	IPS**	07:19:18

*Sequence Aborted
**Improper Sample

Not a Successfully Completed Test Sequence

281

INTOXILYZER 8000
Location: LHCPD
Serial Number: 80-001873
Core Version: 8105.48
07/19/2020

07:20:22

Standard Lot#: 35019100A1
Last Changed By: M. MALOY JR #281

QAS: M. MALOY JR #281
LAKE HAVASU CITY PD

Operator: M. MALOY JR #281
LAKE HAVASU CITY PD

Subject: SQAP,TEST
DOB: 01/01/2000
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	07:21:11
Diagnostic Test	Pass	07:21:46
Air Blank	0.000	07:22:14
0.100 Cal Check	0.098	07:22:34
Air Blank	0.000	07:23:04
Subject Test	0.000*	07:26:11
Air Blank	0.000	07:26:39
Air Blank	0.000	07:27:06
Subject Test	XXX**	07:27:24
Air Blank	RFI***	07:27:32

*Deficient Sample
**Invalid Sample
Begin new deprivation period
***RFI Detect

Not a Successfully Completed Test Sequence

281

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. MALDY #281 AGENCY LNCPD

DATE 07-19-2020 TIME 0730

INTOXILYZER SERIAL # 80-001873 LOCATION LNCPD JAIL

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 099 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Michael Maloy Jr #281

201

INTOXILYZER 8000
Location: LHCPD
Serial Number: 80-001873
Core Version: 8105.48
07/19/2020

07:30:16

Standard Lot#: 35019100A1
Last Changed By: M. MALOY JR #281

QAS: M. MALOY JR #281
LAKE HAVASU CITY PD

Operator: M. MALOY JR #281
LAKE HAVASU CITY PD

Test	g/210L	Time
Air Blank	0.000	07:30:47
0.100 Cal Check	0.099	07:31:07
Air Blank	0.000	07:31:37

Preventative Maintenance
Performed -- Timer Reset