EXHIBIT G-2 THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST, J SOMBROL 187	_AGENCYLNC_PN
DATE 4/23/17	TIME 1430
INTOXILYZER SERIAL# 80-003356	LOCATION MAIN TRIL
Pour a standard alcohol concentration solution simulator. Ensure that a tight seal has been made. 0.2°C 2. Intoxilyzer 8000 display reads "PUSH RUTTON 2	contains a standard alcohol concentration solution AC. OR AC, into a clean dry simulator and assemble the Turn on the simulator and allow temperature to reach 34°C± TO START** dry control test or "W" for wet control test. After selection is
*ONLY COMPLETED 30. DAY G-2 is	DUE TO BEING GONE ON TRAINING

firm

INTOXILYZER 8000 Location: MAIN JAIL

Serial Number: 80-003356 Core Version: 8105.48

04/23/2017 14:30:25

Standard Let#: 22315100A1

Last Changed By: J. SONDROL #187

QAS: J. SONDROL #187

LAKE HAVASJ CITY PD

Operator: J. SONDROL #187

LAKE HAVASU CITY PD

Test	g/210L	Time
Air Blank	0.000	14:31:09
0.100 Cal Check	0.096	14:31:30
Air Blank	0.000	14:32:00

Preventative Maintenance Performed -- Timer Reset