



ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. SANDRALL 187 AGENCY LNC PD

DATE 3/22/17 TIME 2134

INTOXILYZER SERIAL # 80-003356 LOCATION MAIN JAIL

(✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR

Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"  
(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.  
(✓) 4. Air blank completed.  
(✓) 5. Calibration check completed. Test results 0.035 AC.  
(✓) 6. Air blank completed.  
(✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE C. Sandrall 187

INTOXILYZER 8000

Location: MAIN JATT.

Serial Number: 80-003356

Core Version: 8105.48

03/22/2017

21:34:34

-----  
Standard Lot#: 22315100A1

Last Changed By: J. SONDRON #187  
-----

QAS: J. SONDRON #187

LAKE HAVASU CITY PD  
-----

Operator: J. SONDRON #187

LAKE HAVASU CITY PD  
-----

Test	g/210L	Time
Air Blank	0.000	21:35:04
0.100 Cal Check	0.095	21:35:25
Air Blank	0.000	21:35:55

Preventative Maintenance  
Performed Timer Reset

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. RI3-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J SONDRO 187 AGENCY LNC PD  
DATE 3/22/17 TIME 2136  
INTOXILYZER SERIAL # 80-003356 LOCATION MAIN JAIL

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0.096 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J Sondro 187

INTOXILYZER 8000  
Location: MAIN JAIL  
Serial Number: 80-003356  
Core Version: 8105.48  
03/22/2017

21:36:04

---

Standard Lot#: 22315100A1  
Last Changed By: J. SONDROL #187

---

QAS: J. SONDROL #187  
LAKE HAVASU CITY PD

---

Operator: J. SONDROL #187  
LAKE HAVASU CITY PD

---

Subject: SQAP,TEST  
DOB: 01/01/1950  
Sex: M Weight: 123

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	21:36:50
Diagnostic Test	Pass	21:37:25
Air Blank	0.000	21:37:53
0.100 Cal Check	0.096	21:38:14
Air Blank	0.000	21:38:43
Subject Test	0.000	21:39:47
Air Blank	ABT*	21:39:53
Air Blank	IPS**	21:40:01

\*Sequence Aborted

\*\*Improper Sample

**Not a Successfully Completed Test Sequence**

9137

INTOXILYZER 8000  
Location: MAIN JAIL  
Serial Number: 80-003356  
Core Version: 8105.48  
03/22/2017

21:40:43

Standard Lot#: 22315100A1  
Last Changed By: J. SONDROL #187

QAS: J. SONDROL #187  
LAKE HAVASU CITY PD

Operator: J. SONDROL #187  
LAKE HAVASU CITY PD

Subject: SQAP,TEST  
DOB: 01/01/1950  
Sex: M Weight: 123

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	21:41:31
Diagnostic Test	Pass	21:42:06
Air Blank	0.000	21:42:33
0.100 Cal Check	0.096	21:42:54
Air Blank	0.000	21:43:24
Subject Test	0.133*	21:46:31
Air Blank	0.000	21:47:05
Air Blank	0.000	21:47:32
Subject Test	XXX**	21:48:06
Air Blank	RFI***	21:48:13

\*Deficient Sample  
\*\*Invalid Sample  
Begin new deprivation period  
\*\*\*RFI Detect

**Not a Successfully Completed Test Sequence**

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST J. SONDRA 187 AGENCY LNC PD  
DATE 3/22/17 TIME 2149  
INTOXILYZER SERIAL # 80-003356 LOCATION MAIN JAIL

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.  
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.  
 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 095 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

SIGNATURE J. Sondra 187

INTOXILYZER 8000

Location: MAIN JAIL

Serial Number: 80-C03356

Core Version: 8105.48

03/22/2017

21:49:00

-----  
Standard Lot#: 22315100A1

Last Changed By: J. SONDRON #187  
-----

QAS: J. SONDRON #187

LAKE HAVASU CITY PD  
-----

Operator: J. SONDRON #187

LAKE HAVASU CITY PD  
-----

Test	g/210L	Time
Air Blank	0.000	21:49:29
0.100 Cal Check	0.095	21:49:50
Air Blank	0.000	21:50:19

Preventative Maintenance  
Performed -- Timer Reset